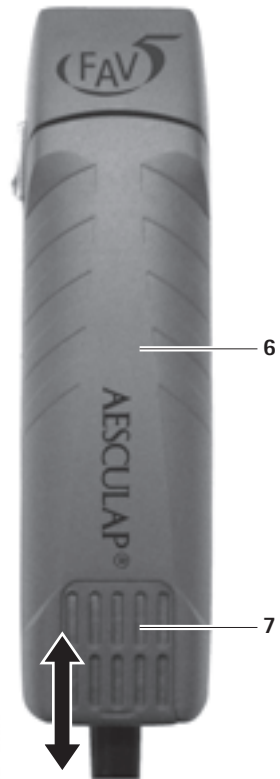
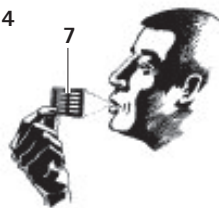
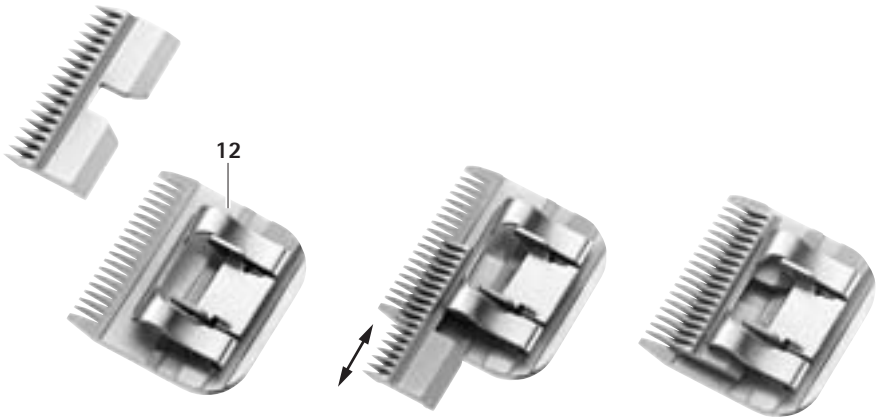
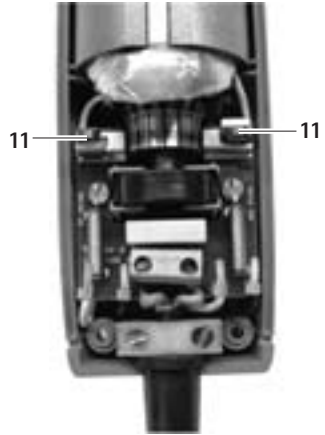
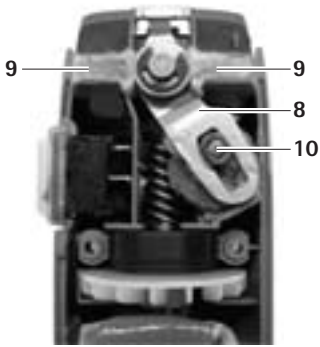
# B. Braun Vet Care

- (GB)** **Instructions for use/Technical description**  
Clipper FAV5
- (D)** **Gebrauchsanweisung/Technische Beschreibung**  
Schermaschine FAV5
- (F)** **Mode d'emploi/Description technique**  
Tondeuse FAV5
- (E)** **Instrucciones de manejo/Descripción técnica**  
Esquiladora FAV5
- (I)** **Istruzioni per l'uso/Descrizione tecnica**  
Tosatrice FAV5
- (NL)** **Gebruiksaanwijzing/Technische beschrijving**  
Scheermachine FAV5
- (RUS)** **Инструкция по применению/Техническое описание**  
Машинка для стрижки FAV5

Clipper FAV5

Legend

- 1 Hinge (complete)
- 2 Shearing head lock
- 3 ON/OFF switch
- 4 Screws (x4)
- 5 Type plate
- 6 Upper housing shell
- 7 Air filter
- 8 Rocker arm
- 9 Oil felts
- 10 Guide piece
- 11 Carbon brush
- 12 Shearing head

Symbols on product

	<p>Adhere to instructions for use</p>
	<p>Labeling of electric and electronic devices according to directive 2002/96/EC (WEEE), see Chapter Disposal</p>

Contents

- 1. Safe handling ..... 3
- 2. Product description ..... 3
  - 2.1 System components ..... 3
  - 2.2 Components necessary for use ..... 3
  - 2.3 Intended use of Clipper FAV5 ..... 3
  - 2.4 Operating principle of Clipper FAV5 ..... 3
- 3. Preparation and setup ..... 4
  - 4. Working with Clipper FAV5 ..... 4
    - 4.1 System set-up ..... 4
      - 4.1.1 Accessories..... 4
      - 4.1.2 Attaching the shearing head ..... 5
    - 4.2 Function checks for Clipper FAV5 ..... 5
    - 4.3 Treatment of the cutting set ..... 6
      - 4.3.1 Sharpening the cutting plate ..... 6
      - 4.3.2 Changing the shearing head..... 6
      - 4.3.3 Lubricating the shearing head ..... 6
- 5. Processing ..... 7
  - 5.1 Cleaning/Disinfecting ..... 7
  - 5.2 Control, care and inspection ..... 7
- 6. Maintenance ..... 8
- 7. Troubleshooting list ..... 9
- 8. Technical Service ..... 10
- 9. Accessories/Spare parts ..... 10
  - 9.1 Shearing heads ..... 11
- 10. Technical specifications ..... 11
  - 10.1 Clipper FAV5 ..... 11
  - 10.2 Declaration of conformity ..... 11
- 11. Disposal ..... 12
  - 11.1 For EU countries only ..... 12
- 12. 2 Years warranty ..... 12

## 1. Safe handling

**DANGER**

**Risk of death by electric shock!**

- **Do not open the product.**
- **Only ever connect the product to power mains with equipment grounding conductor.**

- Prior to use, check for proper condition and functioning of the product.
- Observe our "Notes on Electromagnetic Compatibility (EMC)".
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
  - Use the product only according to these instructions for use.
  - Observe all safety information and maintenance advisories.
  - Ensure the manufacturer's approval before using accessories that are not mentioned in the instructions for use.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

## 2. Product description

### 2.1 System components

Designation	Art. no.
Clipper FAV5	GT105G GT105J GT105K GT330 #10 (USA+UK)
Instructions for use	TA012748

*Note*

*The shearing head GT330 #10 (USA+UK) for Clipper FAV5 is included in the scope of delivery. Various shearing heads (SNAP ON type) can be used for different applications!*

### 2.2 Components necessary for use

- Clipper FAV5
- Shearing head

### 2.3 Intended use of Clipper FAV5

*Note*

*This machine is intended exclusively for cutting animal hair!*

The Clipper FAV5 is used for clipping small animals. The Clipper FAV5 is used with a shearing head to be selected according to the respective application.

### 2.4 Operating principle of Clipper FAV5

The Clipper FAV5 is switched on (off) by pushing the ON/OFF switch up (down).

## Clipper FAV5

### 3. Preparation and setup

Non-compliance with these rules will result in complete exclusion of liability on the part of Aesculap.

- When setting up and operating the product, always observe:
  - national regulations for installation and operation,
  - the national regulations on fire and explosion protection,
  - operating advisories according to IEC-/VDE regulations.
- Prior to connecting the Clipper to mains power, ensure that the voltage specified on type plate 5 matches the mains voltage.

#### Fire and explosion hazard!

- Do not operate the product in explosion-hazard areas.

#### Malfunction of, or damage to the product caused by improper setup!

- Do not expose the product to direct sunlight or moisture.
- Do not cover the ventilation slots of the product.
- Ensure that the mains power socket used for supplying the product is freely accessible.

### 4. Working with Clipper FAV5

#### 4.1 System set-up

##### 4.1.1 Accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

- Please contact your B. Braun/Aesculap Partner or Aesculap Technical Service (address: see Chapter Technical Service) with any inquiries in this respect.

#### 4.1.2 Attaching the shearing head

Risk of injury caused by sharp cutting plates!

- Handle the shearing head of Clipper FAV5 with appropriate caution.

##### Note

The shearing head GT330 #10 (USA+UK) for Clipper FAV5 is included in the scope of delivery. Various shearing heads are available for different applications!

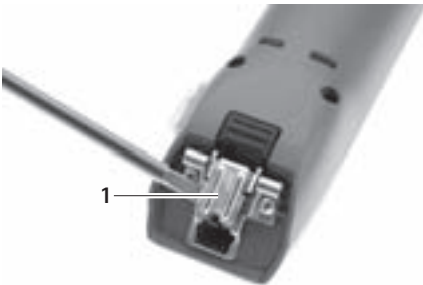
Fig. 1

- If the hinge is closed: Open hinge 1 with a screwdriver, see Fig. 1.

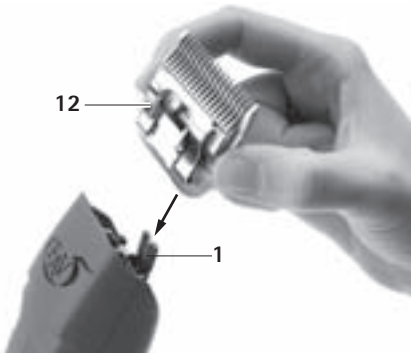
Fig. 2

- Install shearing head 12 on open hinge 1, see Fig. 2.

- Press ON/OFF switch 3.

Fig. 3

- Slightly push shearing head 12 towards Clipper FAV5, see Fig. 3. Shearing head 12 will click into position.
- Check that shearing head 12 is correctly seated on Clipper FAV5.

#### 4.2 Function checks for Clipper FAV5

- Carry out a visual inspection.
- Check the switch position.
- Install the cutting set correctly and lubricate it if necessary, see Chapter Treatment of the cutting set.
- Clean the Clipper FAV5 prior to use, see Chapter Cleaning/Disinfecting.

## Clipper FAV5

### 4.3 Treatment of the cutting set

#### 4.3.1 Sharpening the cutting plate

##### Note

For sharpening the cutting plate, please contact Technical Service, see Chapter Technical Service.

#### 4.3.2 Changing the shearing head

**WARNING**

Risk of injury caused by sharp cutting plates!

- Handle the shearing head of Clipper FAV5 with appropriate caution.

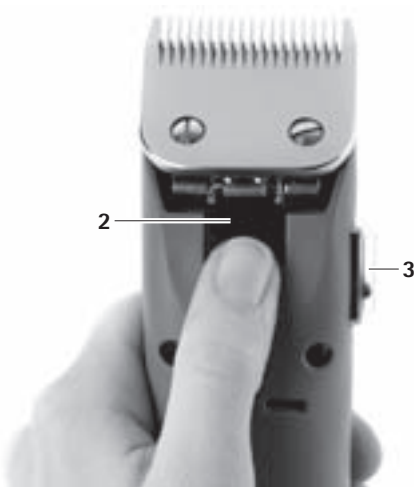
Fig. 4

- Press ON/OFF switch 3 to switch off Clipper FAV5, see Fig. 4.
- Press shearing head lock 2, see Fig. 4.

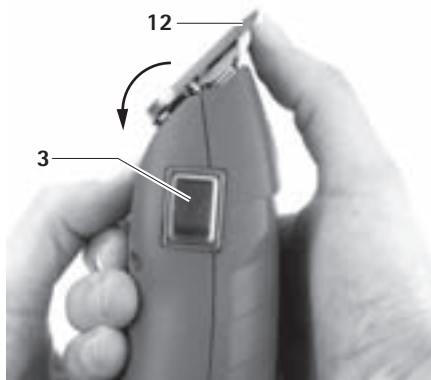
Fig. 5

- Press forward shearing head 12, see Fig. 5.
- Detach shearing head 12.
- Clean open hinge 1 with a brush.
- Install shearing head 12 on hinge 1.
- Plug in the power cord of Clipper FAV5.
- Press ON/OFF switch 3.
- Slightly push shearing head 12 towards Clipper FAV5.  
Shearing head 12 will click into position.
- Check that shearing head 12 is correctly seated on Clipper FAV5.

#### 4.3.3 Lubricating the shearing head

- Clean the upper and lower cutting plates with a soft cloth or brush. Make certain that the gaps between the cutting teeth are cleaned properly.
- Slightly lubricate the upper and lower cutting plates with special lubricant for shearing heads GT604.

## 5. Processing

**CAUTION**

**Damage to, or destruction of the product caused by mechanical cleaning/disinfecting or sterilization!**

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

### 5.1 Cleaning/Disinfecting

*Note*

*Always follow disposal and hygiene instructions and regulations!*

#### Manual cleaning/disinfecting

**DANGER**

**Risk of electric shock and fire hazard!**

- Unplug the unit before cleaning.
- Make certain that no fluids will penetrate the product.
- Leave the product exposed to air for at least 1 minute after cleaning/disinfecting.

**CAUTION**

**Damage to the product due to incorrect cleaning or inappropriate cleaning/disinfecting agents!**

- Use any commercial detergent for surface cleaning.
- Never immerse the product in water or detergent.
- Do not introduce any fluid into the charging bays.
- Clean the contacts in the charging bays, applying extreme care.

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residue of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- If necessary, repeat the cleaning/disinfecting process.

#### Cleaning the air filter

*Note*

*The air filter must be cleaned once a week or whenever necessary!*

- Slide back and remove air filter 7.
- Blow through or wash air filter 7.
- Slide dry air filter 7 into its seating.

### 5.2 Control, care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- Lubricate the shearing head as appropriate, see Chapter Lubricating the shearing head.
- Check the product for any damage, a typical running noise, overheating or excessive vibration.
- Inspect the cutting set for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

## Clipper FAV5

### 6. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product at least once a year.

For services to that end, please contact your national B. Braun/Aesculap agency, see Chapter Technical Service.

#### Service addresses

In the UK

KG PRODUCTS

243-251 City Road, Fenton

Stoke-on-Trent, Staffs

ST4 2PX

United Kingdom,

Phone (44) - 01782 844 866

DIAMOND EDGE

126 Gloucester Rd.

Brighton, BN1 4BU.

United Kingdom

Phone (44) - 01273 605 922

Or in Germany

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 (0) 3681 49820

Fax: +49 (0) 3681 498234

E-Mail: [suhl@aesculap.de](mailto:suhl@aesculap.de)

Other service addresses can be obtained from the address indicated above.

## 7. Troubleshooting list

Malfunction	Finding	Cause	Remedy
Shearing head not moving	Shearing head jammed	Shearing head not properly engaged in its seating	Properly engage the shearing head, see Chapter Attaching the shearing head
		Rocker arm defective	Replace rocker arm
Clipper FAV5 cutting poorly or not at all	Clipper FAV5 not cutting	Cutting edges of shearing head blunted	Have product repaired by the manufacturer
		Animal hair too wet	Only cut dry animal hair
		Shearing head lacking shearing pressure	Have product repaired by the manufacturer
	Cutting performance insufficient	Cutting edges of cutting head not honed correctly	Have product repaired by the manufacturer
	Overheating of shearing head	Lack of lubrication of shearing head edges	Lubricate cutting edges of shearing head, see Chapter Lubricating the shearing head
	Shearing head jammed	Animal hair jammed in shearing head	Demount shearing head and remove animal hair
Motor of Clipper FAV5 not running	Clipper FAV5 not cutting	Debris on electric contacts	Switch off Clipper FAV5 and clean the contacts
		Motor/Electronics defective	Have product repaired by the manufacturer
	Smell of burning from the machine housing	Motor burnt	Have product repaired by the manufacturer
	Shearing head jammed	Debris in shearing head	Clean and lubricate the shearing head, see Chapter Lubricating the shearing head
Motor of Clipper FAV5 running very slowly	Atypical motor noises	Lack of lubrication of shearing head edges	Lubricate cutting edges of shearing head, see Chapter Lubricating the shearing head

## Clipper FAV5

**8. Technical Service**

For service, maintenance or repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

**Service addresses**

In the UK  
 KG PRODUCTS  
 243-251 City Road, Fenton  
 Stoke-on-Trent, Staffs  
 ST4 2PX  
 United Kingdom,  
 Phone (44) - 01782 844 866

DIAMOND EDGE  
 126 Gloucester Rd.  
 Brighton, BN1 4BU.  
 United Kingdom  
 Phone (44) - 01273 605 922

Or in Germany  
 Aesculap Suhl GmbH  
 Fröhliche-Mann-Straße 15  
 98528 Suhl / Germany  
 Phone: +49 (0) 3681 49820  
 Fax: +49 (0) 3681 498234  
 E-Mail: suhl@aesculap.de

Other service addresses can be obtained from the address indicated above.

**9. Accessories/Spare parts**

Art. no.	Designation
GT604	Lubricant
GT605	Grease
GT104614	Carbon brushes
GT104803	Air filter
GT105808	Rocker arm
GT105415	Hinge (complete)

## 9.1 Shearing heads

Shearing head (complete)	Size EU
GT305	0,2 mm
GT310	0,25 mm
GT315	0,5 mm
GT317	0,5 mm
GT320	0,8 mm
GT323	0,8 mm
GT326	1,2 mm
GT330	1,5 mm
GT333	2,0 mm
GT336	2,4 mm
GT339	2,4 mm
GT341	2,8 mm
GT343	3,2 mm
GT345	3,2 mm
GT349	3,2 mm
GT353	2,4 mm
GT357	6,3 mm
GT360	6,3 mm
GT366	9,5 mm

### Note

Detailed descriptions of the shearing heads can be found in brochure V06902.

## 10. Technical specifications

### 10.1 Clipper FAV5

Model	GT105
Stroke rate	2 300 1/min
Protection class (acc. to IEC 60601-1)	II
Weight (without shearing head)	approx. 800 g

### 10.2 Declaration of conformity

	We declare under our sole responsibility that this product conforms to the following standards or other normative documents: 2006/95/EC EC Low Voltage Directive EN 60335-1 EN 60335-2-8 Safety of household and similar electrical appliances Aesculap Suhl GmbH
---	---

## Clipper FAV5

### 11. Disposal

Power tools, accessories and packaging should be recycled through environmentally compatible processes.

#### 11.1 For EU countries only

Do not dispose of power tools with household waste!

According to European Directive 2002/96/EG on waste electric and electronic devices, and its implementation in national legislation, power tools that have reached the end of their service life must be collected separately and recycled through environment-friendly processes.

### 12. 2 Years warranty

Dear customer,

Thank you for choosing one of our products. The Aesculap brand has been trusted for quality and first-class service for many years. Aesculap Suhl GmbH is certified according to DIN EN ISO 9001. We are known to provide innovative, high-performance products for our customers.

Aesculap produces top-quality devices and promises to use only high-grade materials and meticulous manufacturing. As the manufacturer, we undertake warranty covering any quality defects of the product for a period of 2 years from the date of purchase (invoice date).

In addition to your contractual and statutory warranty rights, Aesculap offers manufacturer's warranty on the following terms:

The present manufacturer's warranty will be valid for 2 years from the date of purchase. It applies to faults or defects caused by deficiencies of materials or manufacturing of the product.

The following instances are excluded from our manufacturer's warranty:

Faults caused by improper use, normal wear, inappropriate intervention in the product by unauthorized persons, or assembling of the product using third-party products that do not fit or function properly. Also excluded are defects that do not, or not significantly affect the value or usability of the product.

Should a fault or defect covered by the manufacturer's warranty arise within the warranty period of 2 years, starting from the purchase date, we will either repair or replace, according to our discretion, the faulty or defective product.

Our manufacturer's warranty is subject to receipt of the product and proof of purchase (invoice) by the respective Aesculap national distribution agency or service center.

For addresses of our service centers, see Chapter Technical Service.

Naturally, independent of our manufacturer's warranty, you can still claim warranty rights against the vender, e.g. cure (amendment, repair or replacement), cancellation or price reduction, as far as such claims are within your statutory or contractual rights. These rights are not affected by our additional manufacturer's warranty.

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